Issue Date: 2020.03.11

Version: A/5

## Instructions for use

This product should be handled by professional dental technician, tried-in.

**Product Name:** Fusion Hybrid Ceramic

### Instruction for use:

1. Used for dental restorations, such as full crowns, inlays/onlays and veneers.

### Chemical compositions:

Resin polymer, Inorganic filler, Additive, Coloring agent

### Directions for use:

1. Mill the restoration from Fusion CAD/CAM Restorative with applicable CAD/CAM machine;
2. Grind off sprue with coarse rubber wheel, and finish with medium rubber tool;
3. Pre-polish with fine rubber tool, fine-polish with a soft bristle brush and polishing paste;
4. Pit and fissure staining on the restoration surface, clean restoration in an ultrasonic cleaner or steam cleaner. Gently blow dry with air.

### Shades and Translucencies:

HT(High Translucency): A-D, BL1-BL4, W 21 shades;

HT(High Translucency), Gradient(Multi-chromatic): A-D, BL1-BL4, W 21 shades;

LT(Low Translucency): A-D, BL1-BL4, W 21 shades;

LT(Low Translucency), Gradient(Multi-chromatic): A-D, BL1-BL4, W 21 shades.

**Shelf life:** 5-year is an estimated replacement time. The product is not expected to degrade during this period.

### Storage:

1. Keep in dry place.
2. Store at room temperature.
3. Keep away from intense light or sunlight.
4. Product should be used by the date indicated on the labeling.

**NOTE:** The product shall be used only by trained personnel. The manufacturer is not liable for any erroneous results, since we have no control if this instruction is properly followed or if the product is appropriately used. Any claims for damages therefore shall be limited exclusively to

the commercial value of our products. Please make sure that the dentist has received all of the necessary information for the correct use of the product.

**Medical Device Adverse Event:** so far there is no medical device adverse event on this product.

**Cautions:** Federal Law restricts this device to sale by or on the order of a dentist, please refers to instructional manual.

### Contraindications:

1. Don't use the product if allergic to it;

 2. Contraindicated for patients with parafunctional habits, such as bruxism (hyperfunction) or clenching.

### Safety instruction:

1. Avoid squeezing, shaking or colliding with hard material;
2. This product should only be handled by dental specialist;
3. Wear suitable face mask to avoid fine dust inhalation;
4. Using air blaster in short bursts to avoid fine dust air pollution;
5. There is no limit to temperature during the transportation process;
6. The product is provided no-sterile and is not intended to be sterilized by the end user;
7. The product is supplied to the user in a highly cured state, and this material should not be fired under any circumstances during finishing or build-up/add-on;
8. Single use only;
9. After the product be made of false tooth, it should be cleaned according to the requirements of clinical doctors.

**Disposal:** The material is not hazardous to the environment.

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**Nombre:** MedNet EC-REP GmbH
**Name:** Liaoning Upcera Co., Ltd